

OPLUENZA Massage Chair



Thank you for purchasing our massager products.

This massage chair adopts ergonomic design to realize the functions of neck, back, waist, buttocks, legs, arms, etc A luxurious high-end massage chair.

Before use, please read this manual carefully, and pay special attention to safety precautions to ensure correct operation and use of this product. Keep the manual in a safe place for future reference.

Catalogue

Safety Precautions	1-2
Safety Precautions and Pre-use Precautions	3-4
Maintenance and Repair/Preparation for Use	5
Use safety warnings	6-9
Part Name	10
Function	11
Instructions:	12
1. Turn on the power of the whole machine	12
2. Operation instruction of massage function control	12
1) Shoulder position adjustment	13
2) Menu bar interface	14
3) More automatic mode interface and automatic mode description	14-15
4) Adjust the sitting posture	16
5) Automatic interface	17
6) Manual interface	18-22
7) Voice Assistant	23
8) Health check	24
9) Settings	24
10) Shortcut buttons	25
11) Mobile phone wireless charging slot/ USB link interface	25
12) End the massage	26
Maintenance	27
Troubleshooting	28-29
Product Specifications	30

Safety Precautions

- Please read this [Safety Precautions] carefully before use, and use this product correctly.
- The precautions here indicate the safe and correct use of the product to prevent possible harm or damage to the product, users and others.
- These precautions are distinguished by the two prompts of [Warning] and [Caution] according to the size and severity of the hazard or damage that may be caused by improper operation. Content-related security is of paramount importance and must be followed.

	warn	Incorrect operation may result in serious injury to the user.
	warn	Incorrect operation may result in user injury and property damage. ※Physical damage refers to damages related to houses, property, livestock, pets, etc.
		This logo is to inform the content of warning or attention. (The picture on the left is usually used after adding a warning or attention)
		The mark “⊘” is to inform the prohibited content, and the figure is the description of the specific prohibited content.
		(The picture on the left is forbidden to disassemble)
		This logo is to inform mandatory or required content.

After reading, be sure to keep it where the user can see it at all times.

warn

- If smoke or abnormal noise occurs (abnormal state), please disconnect the power immediately after confirming the safety, and then contact the dealer for repair. It is strictly forbidden to repair by yourself to avoid danger.
- Water enters inside the main unit.
- Sundries fall into the inside of the main unit.
- Heavy falls, falls or body damage.

Notice

- If you feel abnormal during use, stop using it immediately and consult a doctor.
- This product should be placed on a flat ground to avoid injury.
- Before using this product, please confirm whether the power switch is working properly to avoid injury.

Notice



The following persons cannot use:

Thrombosis (blood disease, severe aneurysm, acute venous aneurysm), various skin diseases or skin infections (including subcutaneous tissue inflammation), etc.

The following people should consult a doctor before using a massage chair :

* Patients who are vulnerable to electromagnetic influence and have implanted medical devices in their bodies.

*patients with malignant tumors

*Patients with heart disease.

*Women during pregnancy or menstrual cycle.

* Patients with diabetes and peripheral circulatory disorders.

*Perceptually impaired persons.

* Those who have trauma in the massage area.

Must comply *Fever with body temperature over 38 C , such as acute inflammation (typhoid fever, high blood pressure).

*Severely ill and frail.

*Fractures, acute bone pain patients.

* Muscle sprains, muscle inflammation patients.

*In addition, those with poor constitution, feeling unwell or needing to be quiet, please use it with caution.

*The surface of this product is hot, people who are not sensitive to heat need to pay attention.

This product massage program a cycle of 20 minutes, please avoid continuous massage on the same part. If you need to continue using it, please wait more than 30 minutes to avoid over-stimulating the muscles or nervous system.

Safety Precautions



warn

Disassembly is prohibited



When the body is wet, never use it to avoid electric shock or machine failure.

Must comply



This product cannot be immersed in water or other liquids, nor can it be used and stored in a humid environment such as a bathroom to avoid electric shock or mechanical failure.

Moisture is prohibited




Do this when you are sure there are no people or pets around.

Wet hands are prohibited



It is absolutely forbidden to disassemble or repair this product, power cord, hand controller and other components by yourself, in order to avoid electricity, fire accidents or machine failures. When the power cord is damaged, it is forbidden to disassemble and replace it without permission, and it must be repaired or replaced by the manufacturer or a qualified professional.

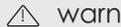

warn

Do not use damaged power cords.

Do not forcibly bend, pull, twist, twist or damage the power cord, and do not hang heavy objects on the power cord.

Using a damaged power cord may result in electric shock, short circuit or fire.

The massage chair will automatically change the angle during the automatic program, and it is forbidden to approach any cracks and gaps to avoid injury.

Do not use a power supply other than AC 220-240V. The power cord should not be wrapped around the product to avoid electric shock, short circuit or fire accident.

Do not let children ride on this product to avoid injury or machine failure.

When adjusting the seat back and seat and adjusting the leg lift, make sure that there are no other people, pets or objects behind and under the leg massager, and during the adjustment process, do not put any part of the body, including hands and feet to prevent injury.


Do not use this product naked to avoid skin damage.

Be sure to use this product for the purpose described in this manual. Do not sit directly on the legs or armrest parts to avoid injury or accident.

When the body cover is disassembled or damaged, do not use it to avoid injury or accident.



Prohibit

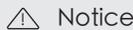

Notice

Must comply



When not in use or when the power is cut off, please turn off the power switch immediately and pull out the power plug, so as to avoid leakage, electric shock or fire accident when the power is turned on.

Wet hands are prohibited



Do not touch the power socket, plug or power cord with wet hands to avoid electric shock.



When cleaning, be sure to turn off the power switch and pull out the power plug from the power outlet to avoid fire or electric shock. When pulling out the power plug, do not pull the power cord forcibly, but pull the bundled head to avoid accidents.



Do not use this product immediately after eating or drinking to avoid physical discomfort.

Do not hang heavy objects on the power cord and manual control line, and the manual control should not be impacted or dropped by external force to avoid malfunction, failure, electric shock or fire accident.

Do not use this product near a fire or heater, and do not use it on a floor with floor heating or other heating. Do not smoke when using to avoid fire.

Safety Precautions

⚠ Notice



Prohibit

- Do not pull hard or scratch the fabric with knives or hard sharp objects to avoid injury or accident.
- Do not forcibly bend or pull the power cord or manual control cord to avoid machine failure or fire accident.
- The hand controller cannot be impacted or dropped by external force, so as to avoid malfunction or machine failure.
- Do not use this product when the power cord, power plug is damaged or the power socket is loose, in order to avoid electric shock, short circuit or fire accident
- When raising and lowering the legs, please do not put your fingers into any cracks to avoid injury.
- Do not forcibly apply external force to the moving parts to avoid personal injury or machine failure.
- Do not use while asleep. Unintentional use may cause injury and physical discomfort.
- Do not drag in the direction of the rollers on wooden floors, carpets or easily scratched floors.
- After moving, place it gently to avoid scratching the ground.

⚠ Notice



Prohibit

- This product can only be used in general household places, please do not use it in other places.
- Do not cover the product with a blanket, etc., to avoid fire accidents caused by the high temperature inside the machine.
- Please make sure that the power plug and the power socket are well matched, and there is no garbage attached. Do not use the terminal block to avoid electric shock, short circuit or fire accident.
- Do not stand, step on, or stack heavy objects on the body or leg parts to avoid personal injury or machine failure.
- Do not use this product near the fire source or flammable materials to avoid product deformation, machine failure or fire accident.
- Do not use this product in an environment with high floating concentration of oil and gas and other chemicals to avoid product deformation, machine failure or fire accident. Do not wipe or spray this product with organic solvents, alcohol or insecticides to avoid cracks in the machine or electric shock or fire accident.
- During use, do not let children or mentally handicapped people approach to avoid injury accidents.
- Paralyzed people cannot use this product to avoid physical discomfort or injury.



Before use

1. Check the surroundings before use

Make sure that there are no people, pets and other objects around the unit.

2. Make sure that there are no foreign objects between the gaps of the machine

Before massaging, please make sure that no foreign objects are caught in the calf, feet, etc.

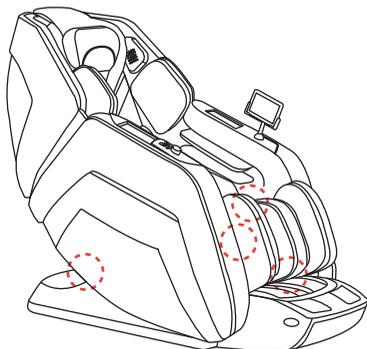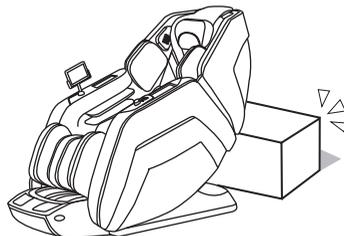

⚠ Notice

- Do not put your hands, feet and head into the gap between your calf, seat and armrest.

Before taking a seat, please make sure that there are no foreign objects between the various parts of the machine. Do not put your hands, arms or head between the backrest and the back cover. Do not place pets on the massage chair (seat, back, armrests, legs, etc.).

Before use

3. Turn on the power

Before massaging, please make sure that no foreign objects are caught in the calf, feet, etc.


 warn

- Please check whether the power cord and power plug are damaged before use.
- Before taking a seat, please make sure that there are no foreign objects between the various parts of the machine. Be sure to insert the power plug all the way. Otherwise, electric shock or overheating may result in fire.
- Please use 220-50Hz AC. Do not use this product abroad or connect the transformer without authorization. Otherwise, electric shock, machine failure, or overheating may result in a fire. Do not insert or unplug the power plug with wet hands.



Insert the power plug into the power outlet.

Press the power switch at the bottom of the machine to "I".

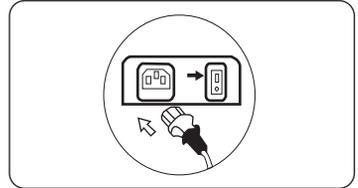


4. Make sure the massage wheel is in the storage position

When the massage wheel is not in the storage position, press the "standby button" to return the massage wheel to the storage position.



Press the "standby button" to massage back to the storage position.

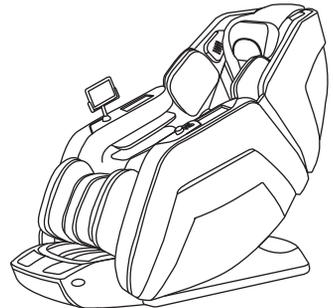


5. Confirm that the fabric is not damaged and there are cracks

Key check area: The fabric of the backrest covering the range of motion of the massage wheel.

 Notice

- Before using this machine, be sure to open the cushion first, and check whether the fabric in the key area shown in the figure is damaged. Other parts should also confirm whether the fabric is damaged. (No matter how small the damage is found, please stop using it immediately, unplug the power plug, and send the machine for repair. Using the machine with damaged fabrics may result in the risk of injury or electric shock.

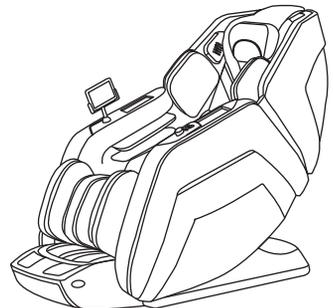

Maintenance and Repair/Preparation for Use

Product Care and Repair


1. This product can only be repaired by the service center designated by the manufacturer, and the user cannot disassemble and repair it by himself.
2. Please turn off the main power switch after use.
3. When not in use for a long time, please roll up the power cord of this product and store it in a dry and dust-free environment.
4. Do not store in high temperature or near an open flame, and do not expose to direct sunlight for a long time.
5. Please use a dry cloth to clean the utensils. Do not use thinner, benzene or alcohol.
6. Please use it intermittently, and the product should not be allowed to run continuously for a long time.

Mobile massage chair

As shown in the figure on the right, tilt the chair back about 45° , and use the casters to push it forward or backward carefully until it is put down at a suitable position.


Notice!

- When someone is lying on the massage chair, please move the massage chair.
- When moving the massage chair, the floor may be damaged, so it is recommended to put a cushion under the product.
- To avoid falling, please hold the massage chair firmly when moving.

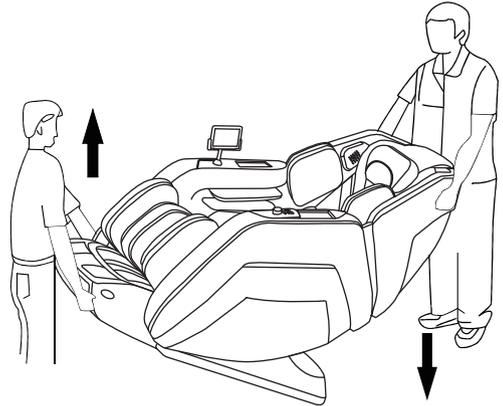

Place the massage chair

When placing the massage chair, be sure to leave enough space around the chair so that the legs and back of the chair can rise and fall freely. The recommended placement space is shown in the picture on the right.

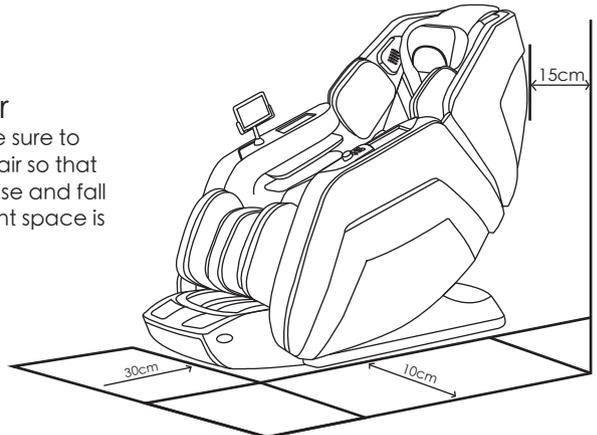


Turn on the power of the whole machine

Plug in the power cord, pull the power switch to the "I" position, the switch indicator lights up, indicating that the power is turned on.

Use safety warnings


▲ Warning

When the calf frame of the massage chair rises, the person shall not exert gravity on the calf frame to prevent the massage chair from falling forward, so as to avoid product damage or personal injury

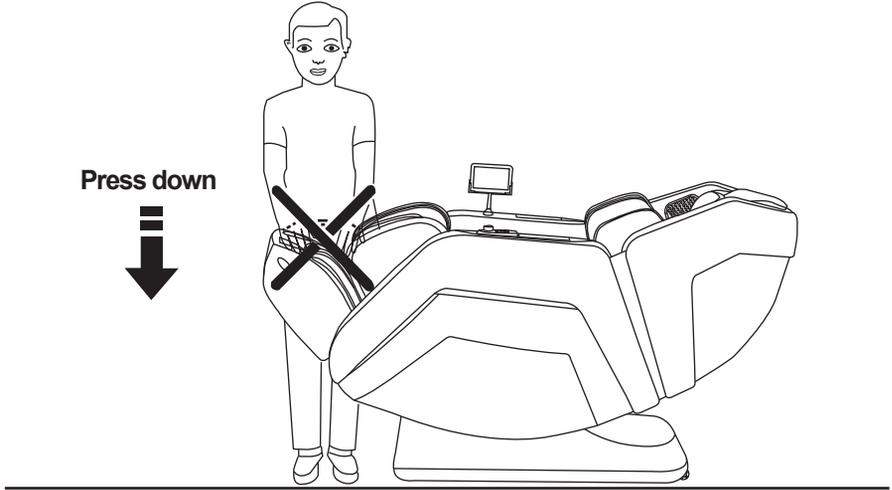

▲ Warning


Before sitting on the massage chair, it is necessary to confirm that the massage manipulator cannot stay in the position of the cushion, so as to avoid discomfort to the buttocks and damage to the massage manipulator during the downward pressure of the human body.

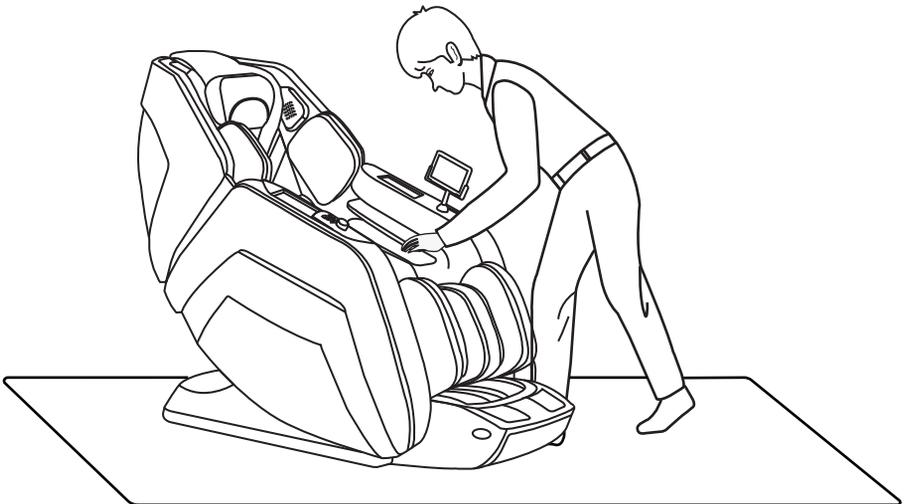

Use safety warnings


▲ Warning

In standby or power-on state, do not remove or pull the power cord to avoid product damage or accidental electric shock; it is recommended to unplug the power cord and store it when the massage chair is not in use.

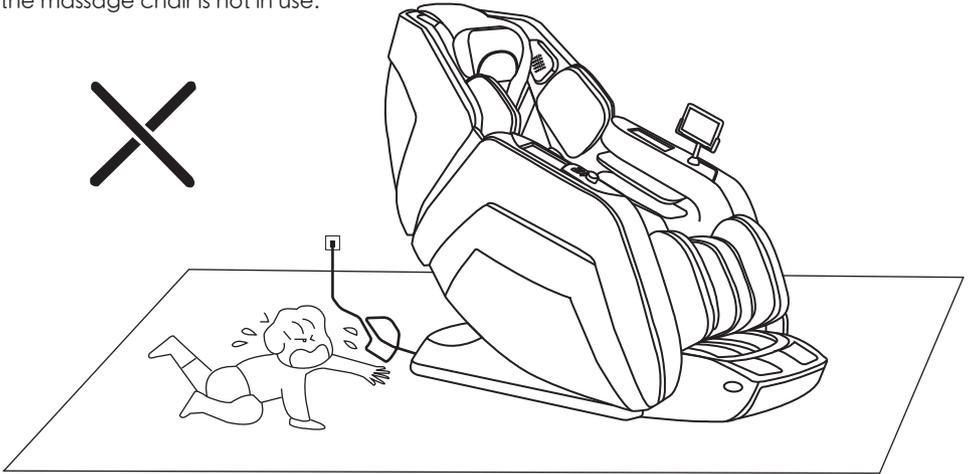

▲ Warning


When turning on the zero-gravity reclining function, be sure to pay attention to whether there are children and pets staying or playing under the massage chair at the bottom of the calf frame and under the backrest frame to avoid product damage and personal injury.

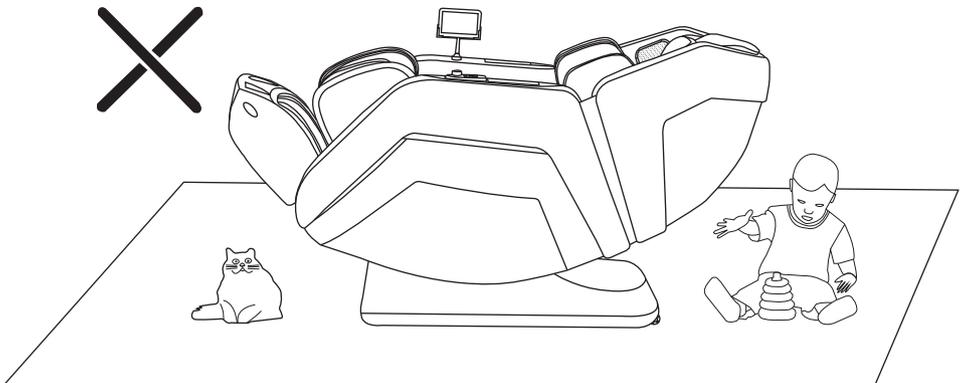

Use safety warnings


▲ Warning

Before using the massage chair, please remove the hard objects in the pocket or carry-on jewelry before massaging (such as keys, watches, etc.) to avoid product damage or personal injury during the massage. harm.

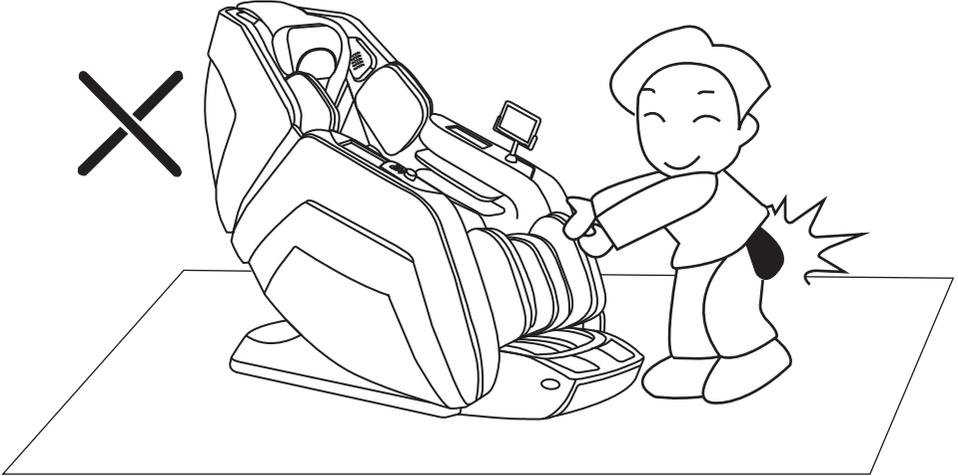

▲ Warning


Do not unplug the power cord during the massage, or directly turn off the power switch behind the massage chair to avoid product damage or personal injury.

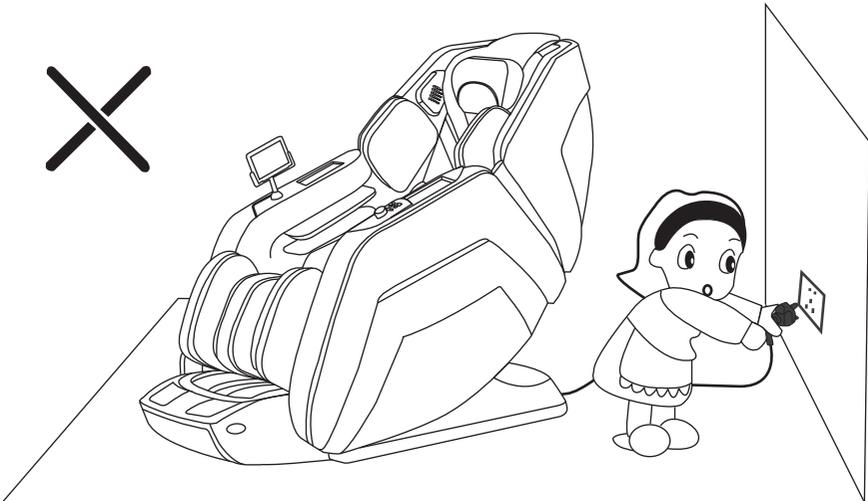

Use safety warnings


▲ Warning

Please do not fall asleep on the massage chair during the use of the massage chair to avoid personal injury caused by unconscious use.

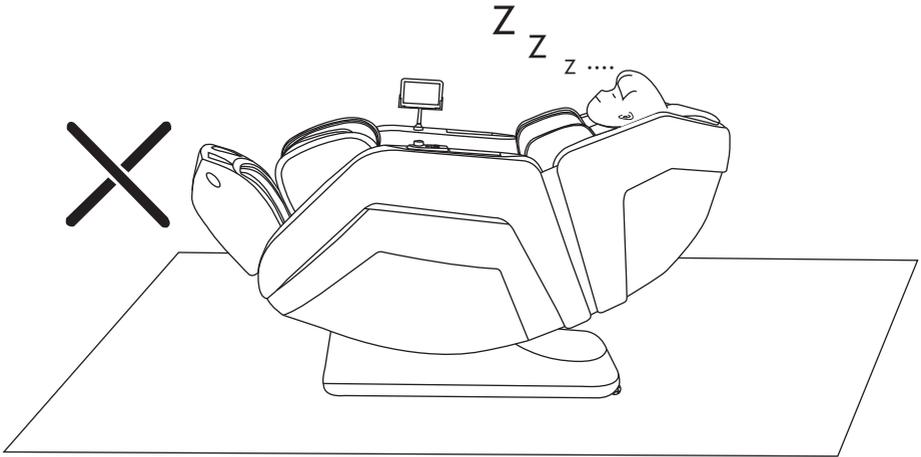


▲ Warning


When using the massage chair, please do not exceed two people to avoid product failure or personal injury. In addition, do not let children play on the massage chair, or climb on the calves, seat cushions, feet, backrests or armrests, otherwise they may cause damage, cause an accident or cause injury.

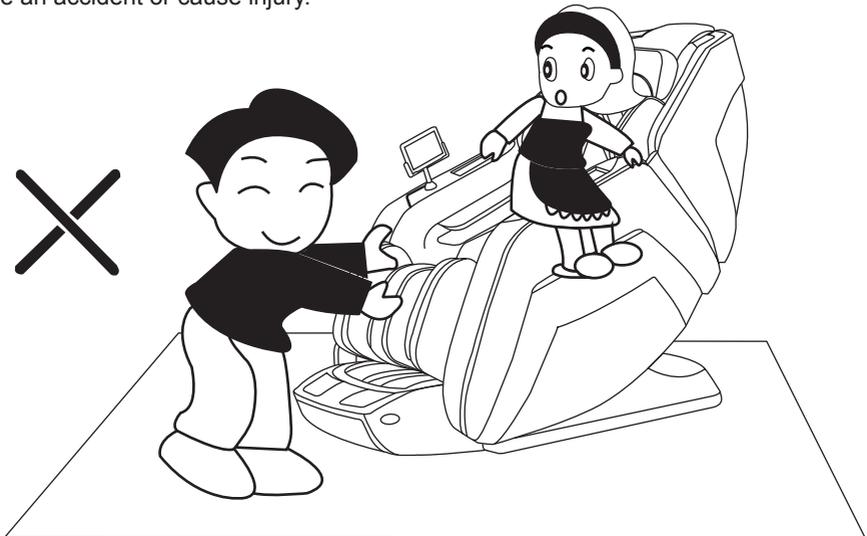


Part Name

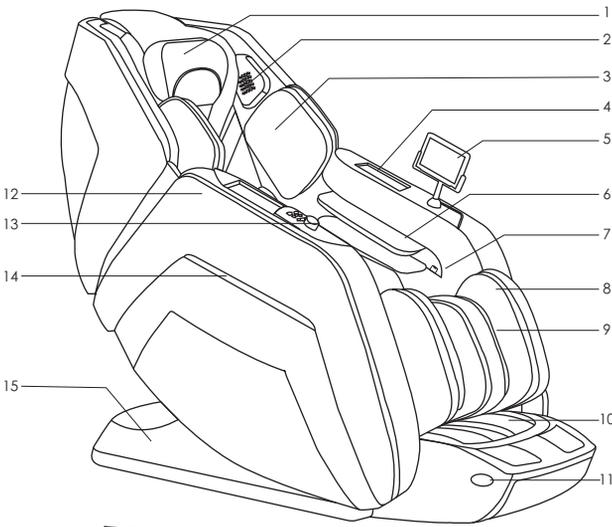

- 1. Head cushion
- 2. 3D digital audio
- 3. Upper arm air pressure components
- 4. Mobile phone wireless charging slot/usb charging port
- 5. Large screen controller
- 6. Arm air pressure components
- 7. Health detection components
- 8. Leg air pressure components
- 9. Leg rubbing components
- 10. Foot air pressure and roller assembly
- 11. Foot decorative lights

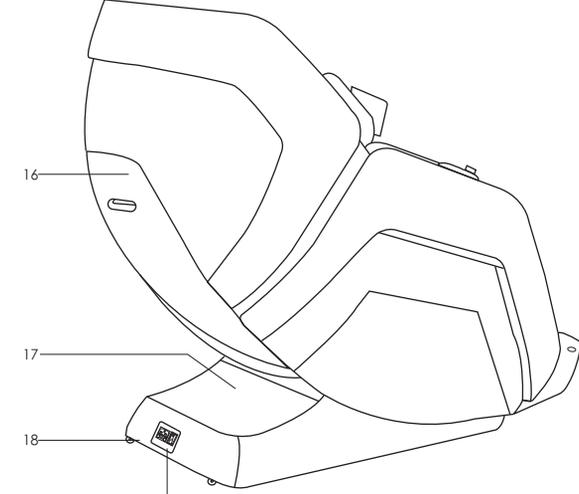

- 12. Armrest
- 13. Shortcut Controller
- 14. Decorative lights
- 15. Side cover
- 16. Backrest cover
- 17. Back cover of power box
- 18. Mobile pulley

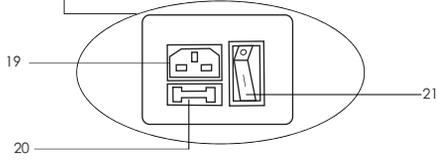


- 19. Power input
- 20. Power fuse
- 21. Power switch

Function


- This product is designed with a set of 4D manipulators that can walk up and down, stretch back and forth, and have a silent design.
- Designed with automatic shoulder position detection and micro-adjustment functions; automatic detection of human body curves and massage acupressure points, which automatically adjusts the amount of manipulator forward and backward expansion according to the detected human body curves and massage acupressure points, making the massage more humane and scientific.
- Equipped with characteristic automatic massage functions: "National Health Care", "Queen's Body", "Neck and Shoulder Care Mode", "Lower Back Soothing Mode", "Beautiful Hips and Waist Care Mode", "Thai SPA Mode", "Soothing Mode" Ten types of automatic massage functions, including tendon stretching mode, airbag massage mode, rocking sleep mode and vitality morning mode.
- Designed with manipulator manual massage range selection: including fixed point, upper back, lower back, partial, whole and other five types of massage range selection.
- Designed with manual massage function: including stop, kneading, tapping, kneading and tapping, shiatsu, tapping and other eight massage techniques to choose from.
- Designed with massage adjustment function: including 4D intensity adjustment in 6 grades, air pressure intensity adjustment in 6 grades, and time adjustment in 4 grades. There are three types of massage adjustment function options.
- Designed with upper body air pressure massage function: shoulder air pressure massage function (built-in 4 airbags), arm air pressure massage function (built-in 16 airbags), waist buttocks air pressure massage function (built-in 8) six-speed air pressure intensity adjustment.
- Designed with lower body air pressure massage function: calf air pressure massage function (built-in 24 airbags), six levels of air pressure intensity adjustment.
- The backrest frame uses sliding rail technology to lie down, so that the backrest is 10 cm away from the wall, saving space to the greatest extent.
- The backrest frame rail adopts the curve rail technology, which completely fits the curve of the back of the human body, and can enjoy a close-fitting massage from the head to the thigh.
- There are bump massage and magnetic therapy functions in the left and right palm airbags.
- Design with back heating function: using far-infrared carbon fiber heating source.
- Designed with a health detection system, which collects cardiac bio-telecommunication signals through the principle of ECG measurement, detects blood oxygen, blood pressure, heart rate and other health tests. The test data is for reference only and cannot be used as a basis for medical diagnosis.
- Designed with armrest portable wireless charging slot and USB charging port.
- Music playback system, built-in 3D digital audio on the left and right. You can use the Bluetooth function of smart devices to connect wirelessly to play music.
- Designed with negative ion function to release negative ions and purify the surrounding air.
- Designed with independent lifting adjustment function of calf frame, linked lifting adjustment function of backrest frame, and automatic lying angle adjustment function of zero gravity.
- It is designed with calf rubbing function, airbag swinging function, and the three-speed intensity can be adjusted.
- Designed with foot sole roller massage function, the scraping roller rolls to massage the toes and feet, and the rolling scraping massage to the arch of the foot; the speed can be adjusted in three gears.
- The beautiful legs are equipped with an inductive telescopic adjustment function. The extension length of the feet is determined according to the length of the user's legs. Position the heel of the user's feet to adjust the distance between the beautiful legs.
- Automatic sit-up function when power off, automatic reset function of manipulator.
- Equipped with TFT touch control screen, shortcut operation buttons and AI voice system for easier operation.

Instructions


1. Turn on the power of the whole machine

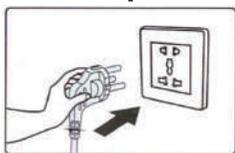

①. Schematic diagram of power on

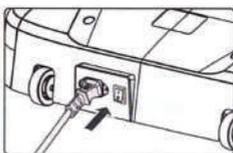

②. Schematic diagram of the connection line of the whole machine

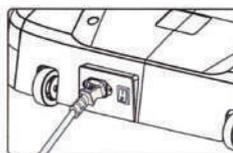

③. Schematic diagram of the power switch of the whole machine


2. Massage function control operation

1. Standby

- After the massage chair is powered on and off, the black screen of the hand controller can be turned on, sit up and lie down in standby mode, and you can also use the voice-activated wake-up button.

2. start massage

- Press the key to start the massage, the backrest frame and the calf frame will automatically lie down to the set angle.
- After the automatic lying in place, the massage chair will automatically detect the position and shape of the human body and wait for the detection to be completed. If you skip the shoulder position and body shape detection directly, you can start the massage directly with the default value.

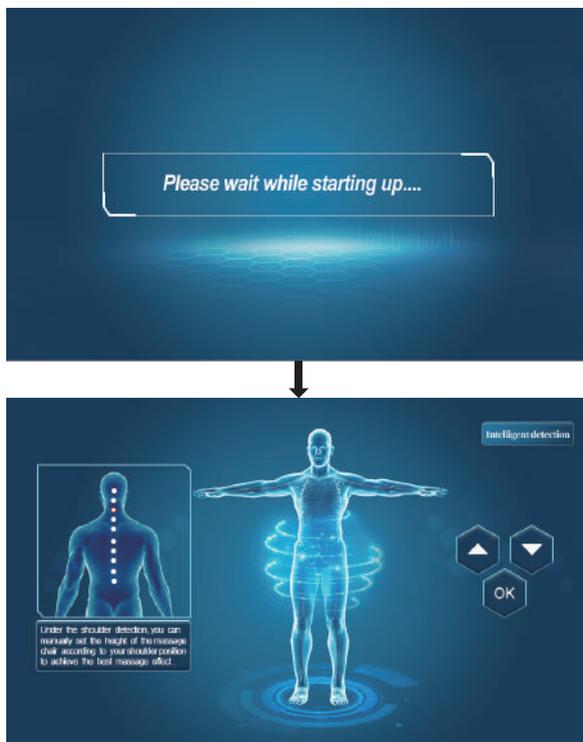

Instructions


1) Shoulder position adjustment

After detecting the shoulder position, the buzzer will prompt "DiDi". If it is different from the actual shoulder position, adjust it to the appropriate position by pressing the "Up button" or "Down button". A total of 10 adjustable shoulder positions. If there is no shoulder position adjustment within 10 seconds, it will automatically enter the massage function. If you directly press the "confirm button" to enter the massage immediately. If the shoulder position is not detected, the default shoulder position height is used for massage.




Button	Description
	Adjust the position of the massage ball upward to make it reach the appropriate shoulder position.
	Adjust the position of the massage ball downward to make it reach the appropriate shoulder position.
	

After entering the massage program, if you need to adjust the shoulder height adjustment, on the automatic running status information page, you can directly select the "shoulder height adjustment" button to enter the intelligent detection interface again, and adjust the shoulder position and height through "▲" or "▼" press "OK" to return to the automatic menu interface.

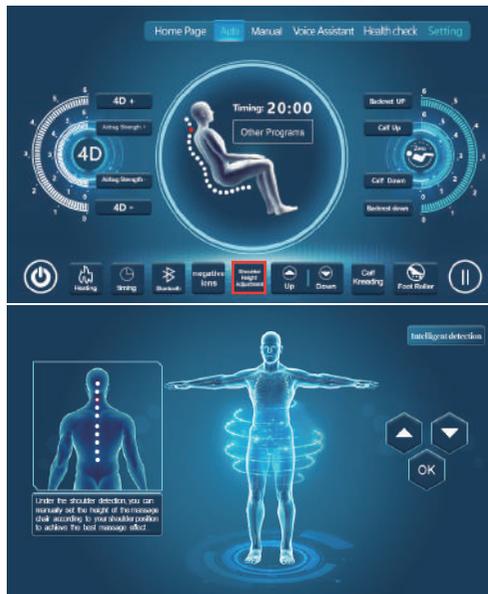

Instructions

2) menu bar interface

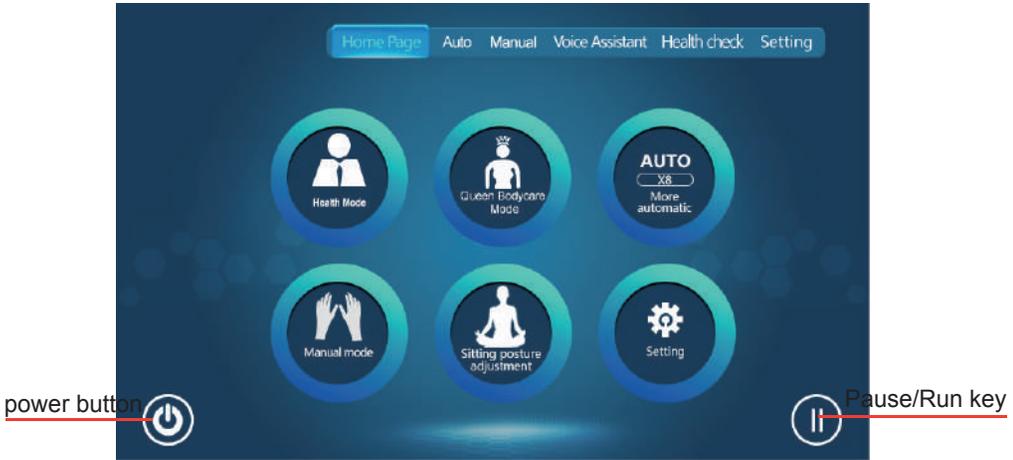

Menu bar interface: "Home", "Auto", "Manual", "Voice Assistant", "Health Check", "Setting" is the function menu bar.

Homepage interface: It mainly consists of the automatic mode of national health care, queen body beauty mode, more automatic modes, manual mode, adjustment of sitting posture and setting menu to form the main page, which is convenient for users to operate

3) More Auto Mode Interfaces




More interface: According to the user's physical condition and massage time, you can choose "Health Mode", "Queen Bodycare Mode", "Shoulder Care Mode", "Back Relieve Mode", "Buttock and Waist Mode", "Tai SPA Mode", "Muscle Relax Mode", "Airbag Massage Mode", "Shake Sleeping Mode", "Fresh Morning Mode" One of the eleven massage programs of "Type" and "Manual Mode" can be used for targeted massage.

Instructions

Description of automatic mode

Function	Description
Health Mode	Various massage methods such as manipulators, airbags and rollers work together, and with Bluetooth music of your choice, you can relax your body and mind.
Queen Bodycare Mode	Kneading massage for neck and shoulders with shoulder airbags, kneading massage for back and waist, kneading for arms, kneading and patting massage for thighs, and simultaneous massage by opening foot rollers, leg airbags, and arm airbags.
Shoulder Care Mode	The unique massage technique for the shoulders and neck helps to promote blood circulation, relieve neck and shoulder pain, and improve blood supply to the brain.
Back Relieve Mode	The massage manipulator slowly slides down from the shoulders and neck to the waist, and then massages from the waist to the neck and shoulders by kneading and patting. Then massage from the neck and shoulders to the waist with tapping and percussion massage techniques, and then slowly slide up from the waist robot to the shoulders and neck. Massage over and over again.
Buttock and Waist Mode	For prolonged sitting in the office, massage can increase the amount of physical activity and improve body function.
Tai SPA Mode	Open the shoulder air bag, the backrest slowly descends, and the leg air bag and the sole roller are opened at the same time, the calf slowly descends, and the manipulator massages to the waist (airbag manipulator inflates and deflates the waist) to realize a slow one-on-one-off waist. Open the arm air bag, and the mechanical hand beats from the waist to the thigh, and then massages from the thigh to the neck and shoulders. loop action.
Muscle Relax Mode	To clear the meridians, quickly improve the body's microcirculation, enhance immunity.
Airbag Massage Mode	Through airbag pressing, rubbing, and roller scraping, it can relieve the soreness and swelling of the shoulders, arms, knees, calves and other key parts after exercise and exercise, and quickly recover the physical fatigue after exercise.
Shake Sleeping Mode	With kneading and shiatsu massage techniques, combined with low-intensity air pressure massage, the massage of the twelve major health acupoints helps the body to adjust and balance; at the same time, the massage chair automatically swings back and forth to form a zero-gravity cradle massage state, making people more able Fall asleep quickly, improve sleep quickly, improve sleep quality.
Fresh Morning Mode	Gentle and comfortable deep massage, relieve physical fatigue, relax the body, the most comfortable massage experi-

4) Adjust your sitting posture

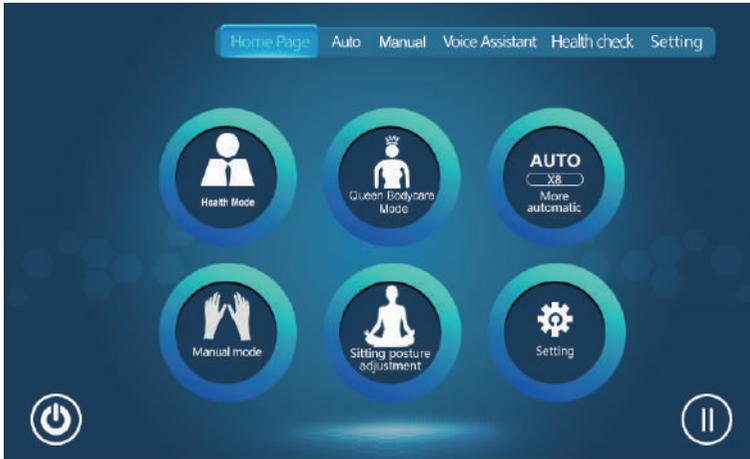

Adjust the sitting posture: According to the user's lying comfort, you can enter to adjust the sitting posture.



Sitting posture adjustment: According to the user's sitting posture adjustment, you can choose G1\G2\G3 sitting posture, or you can control the arrow mark by manual control, adjust the backrest lift, calf lift, leg extension and other free adjustment massage chairs, massage chairs can be adjusted to a more comfortable sitting position, after the adjustment After sitting, press "⌂" key to return to the main page.

Instructions

5) automatic interface



Press the "menu bar" to enter the automatic massage function interface. The automatic interface includes 4D manipulator, airbag strength, zero gravity, backrest lift, calf lift, back heating, timing, Bluetooth, negative ions, shoulder height adjustment, leg rub, foot roller, various Various massage modes and other functional interface control.

The automatic interface can customize the massage mode according to the user's needs: you can choose "National Health Care", "Queen Body", "Neck and Shoulder Care Mode", "Lower Back Soothing Mode", "Beautiful Hips and Waist Care Mode", "ThaiSPA mode", "Stimulating and stretching mode", "Airbag massage mode", "Swing sleep mode", "Energetic Morning Mode" can be used for targeted massage with any one of the ten massage programs.

Under the automatic interface, individual functions of the massage chair can be turned on according to the user's needs to customize the massage:

4D+/- key: Press "4D+" to control the manipulator to push out, press "4D-" to control the manipulator to lower, 6-speed massage, Overhead height adjustment.

Airbag strength +/- key: press "airbag strength +" to adjust the airbag strength, press "airbag strength -" to adjust, The strength of the airbag is weak, and the strength of the 6-speed airbag can be adjusted freely.

Zero Gravity: When the "Zero Gravity" is turned on, the massage chair automatically adjusts to the zero gravity state, and then press the "Zero Gravity" button to turn off the massage chair and return to the initial state.

Backrest up/down button: Long press the "Backrest Down" button to lower the backrest; long press the "Backrest Raise" button to raise the backrest.

Calf up/down button: long press the "Calf Down" button to lower the calf; press the "Calf Up" button to raise the calf.

Instructions

- Heating:** Turn on/off the back heating function.
- Timing:** massage time 15 minutes/20 minutes/25 minutes/30 minutes timed massage settings, click to cycle to select massage time.
- Bluetooth:** Turn on/off the bluetooth function. When turned on, the massage chair's bluetooth function can link the sound source device with bluetooth function (such as mobile phone, MID tablet, etc.) It is wirelessly transmitted to the massage chair sound system through Bluetooth for music playback.
- Negative ions:** turn on/off the negative ion function, release negative ions, purify the surrounding air.
- Shoulder height adjustment:** After entering the massage program, if you need to adjust the shoulder height adjustment, click the "Shoulder Height Adjustment" button, re-enter the intelligent detection, and adjust the shoulder position and height through "▲" or "▼"; press "OK" to return to the automatic menu interface.
- Up and down:** adjust the position of the massage manipulator, targeted massage.
- Leg rubbing:** The leg rubbing function can be adjusted to three levels of intensity.
- Foot roller:** The leg roller function can be adjusted in three gears.

6)Manual interface



Press the "menu bar" to enter the manual massage interface, which can be freely set according to the user's massage needs. The manual interface includes: "massage technique", "massage width", "massage speed", "massage part", "airbag strength", "airbag strength" Parts, "Kneading Legs", "Foot Roller", "4D Robot", "Robot Up and Down", "Heating", "Timing", "Bluetooth" and other functions make up the interface.

- Heating:** Turn on/off the back heating function.
- Timing:** massage time 15 minutes/20 minutes/25 minutes/30 minutes timed massage settings, click to cycle to select massage time.
- Bluetooth:** Turn on/off the Bluetooth function. When turned on, the Bluetooth function of the massage chair can link the audio source device with Bluetooth function (such as mobile phone, MID tablet, etc.) and the Bluetooth module of the massage chair. After pairing and connection, the music played by the audio source device can be wirelessly transmitted to the massage chair speaker through Bluetooth. The system plays music.

Instructions

●**4D+/- key:** Press "4D+" to control the manipulator to push out, press "4D-" to control the manipulator to descend, and the height of the 6-speed massage head can be adjusted freely.

●**Manipulator up/down:** long press "Up" to adjust the massage manipulator to go up, long press "Down" to adjust the massage manipulator to go down, targeted massage.

① Manual-massage technique



Massage techniques: You can manually control the six massage techniques of "Kneading", "Massage", "Tap", "Knock", "Knead and Knock" and "Shiatsu" to switch massage or combine massage;

② Manual - massage width



Massage width: In the manual massage mode, you can manually adjust the width/medium/narrow of the robotic massage to achieve the best state of your body massage.

Instructions

③ Manual - massage speed



Massage speed: In the manual massage mode, you can manually adjust the massage speed of the manipulator in 6 gears to meet the massage speed requirements of your body.

④ Manual - massage part



Massage part: In manual massage mode, you can manually adjust the "whole body", "partial" and "fixed point" of the massage part of the manipulator; long press "up" or "down" to adjust the massage part.

Instructions

⑤ Manual - airbag strength



Airbag strength: According to the user's needs, the airbag strength can be manually adjusted from 1 to 6 gears.

⑥ Manual-airbag part



Airbag part: According to the user's needs, the airbag part can be manually controlled. Massage: "Shoulder", "Hand", "Waist/hip", "Leg", "Foot", "Body" to switch Airbag massage or combination. Partial airbag massage.

Instructions

⑦ Manual - leg rub



Leg rubbing: According to the user's needs, the leg rubbing speed can be adjusted manually: 1-3 speed rubbing massage and closing function can be selected.


⑧ Manual - Leg Roller



Foot roller: according to the user's needs, you can manually adjust the foot roller massage speed: 1-3 speed massage and roller off function can be selected.

Instructions

7) Voice Assistant

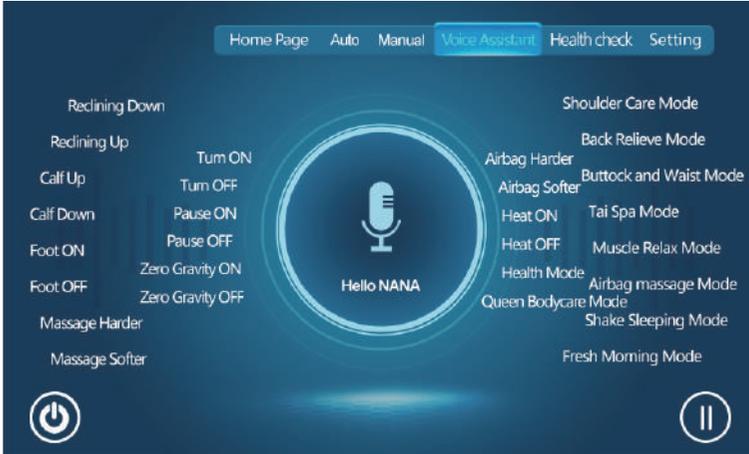

You can perform voice control operations according to the "Message Voice Entry" on the voice assistant, or press the "G" key to turn on or off the voice assistant. Note: When the intelligent voice control is turned off, voice wake-up cannot be performed.

Wake Word: Hello!NANA

Turn ON

Turn OFF

Pause ON

Pause OFF

Zero Gravity ON

Zero Gravity OFF

Reclining Down

Reclining Up

Calf Up

Calf Down

Foot ON

Foot OFF

Massage Harder

Massage Softer

Airbag Harder

Airbag Softer

Heat ON

Heat OFF

Health Mode

Queen Bodycare Mode

Shoulder Care Mode

Back Relieve Mode

Buttock and Waist Mode

Tai Spa Mode

Muscle Relax Mode


Airbag massage Mode

Shake Sleeping Mode

Fresh Morning Mode

Instructions

8) Health check

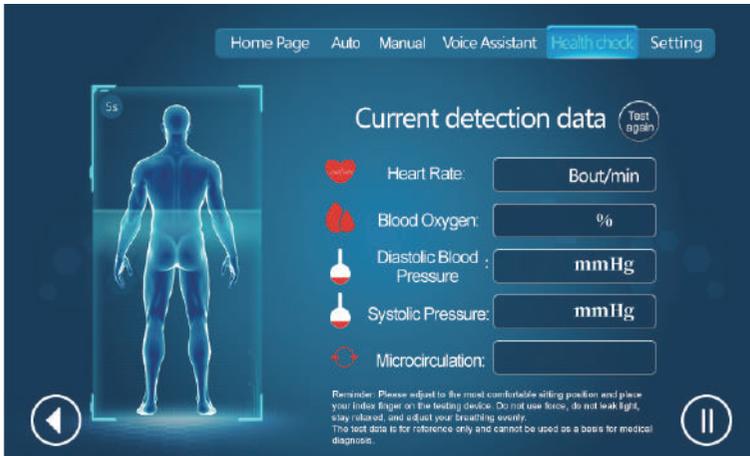

Health detection: Please adjust to the most comfortable sitting position, and place your index finger on the detection device (note that the detection device is massaged on the right hand), do not use force, do not leak light, keep relaxed, and adjust your breathing evenly. Test heart rate, blood oxygen, diastolic blood pressure, systolic blood pressure, and microcirculation data are for reference only and cannot be used as a basis for medical diagnosis. If you need to check again, click "Test again".


9) Settings



The "Settings" interface can set the massage chair according to user needs: massage time, key tone, ambient light, language.

Instructions

10) Shortcut buttons

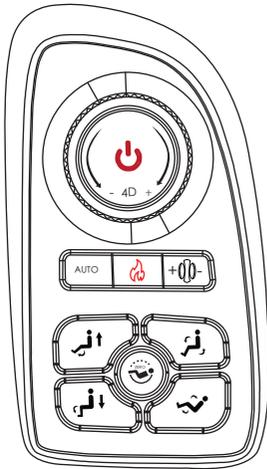

Shortcut Controller

- **ON/OFF key**  : long press the “  ” key to turn on/off the massage function of the whole machine, press the key “  ” once to pause or run the massage function of the whole machine.
- **4D+/- key:** Turn "4D+" to control the ejection of the manipulator, turn "4D-" to control the lowering of the manipulator, and adjust the height of the massage head ejection in gears 0-6.
- **AUTO:** According to the user's physical state and massage time, you can click the "AUTO" button to switch between ten massage programs: "Health Mode", "Queen Bodycare Mode", "Shoulder Care Mode", "Back Relieve Mode", "Buttock and Waist Mode", "Tai Spa Mode" "Muscle Relax Mode", "Air massage Mode", "Shake Sleeping Mode", "Fresh Morning Mode" to switch modes.
- **Heating**  : Turn on/off the back heating function.
- **Airbag** +00- : Press "+00-" to adjust the airbag strength from strength to weakness, a total of 6 levels of strength are cycled in turn.
- **Zero Gravity**  : When “  ” is turned on, the massage chair automatically adjusts to the zero gravity state, and then press the “  ” key again to close the massage chair and return to the initial state of power-on.
- **Backrest** انفر / انفر key: long press the “ انفر ” key, the backrest rises; long press the “ انفر ” key, the backrest falls.
- **Calf** انفر / انفر key: long press the “ انفر ” key, the calf goes down; press the “ انفر ” key, the calf rises.

11) Mobile phone wireless charging slot/USB link interface




On the right armrest of the massage chair, there is a wireless charging device and a USB link interface, which is convenient for users to experience.


Instructions

12) End the massage


During the massage, press the power switch to immediately turn off all massage functions, reset the backrest frame and calf frame, or when the massage time is up, the massage will automatically turn off all massage functions, and the backrest frame and calf frame will not reset.

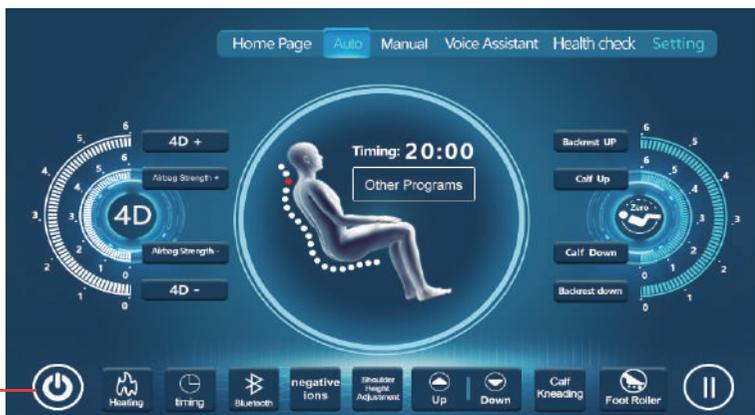


Power switch key

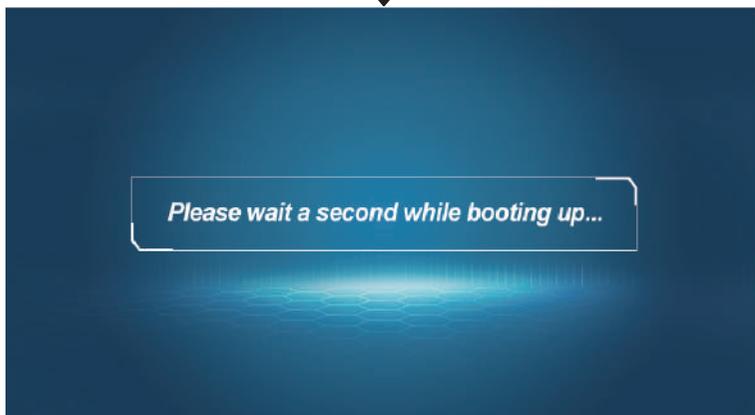


13) Cut off the power supply of the whole machine, as shown in the attached picture (schematic diagram of cutting off the power supply of the whole machine)

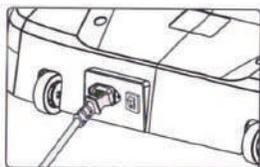

① Schematic diagram of turning off the power of the whole machine

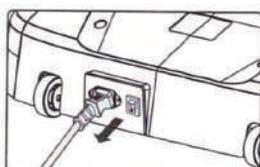

② Unplug the connection diagram of the whole machine

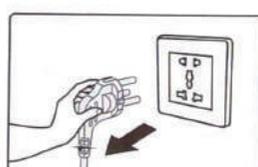



③ Schematic diagram of unplugging the power plug

Maintenance

Clean the synthetic leather part

Wipe the leather with a soft, dry cloth.

- Never use chemicals such as thinner, gasoline, alcohol, etc. to clean the unit.
- When using commercially available leather care products (rags), please follow the instructions for use.
- If synthetic leather is particularly dirty, remove the
 - ① Dip a soft cloth in water or hot water containing 3% to 5% neutral detergent, and wring it out thoroughly.
 - ② Wipe the leather surface with the above soft cloth.
 - ③ Wipe off the cleaning solution with a cloth soaked in clean water and thoroughly wrung out.
 - ④ Wipe with a wrung soft cloth.
 - ⑤ Let it air dry naturally.
- When it is difficult to remove the stains, please soak the commercially available melamine foam in neutral detergent, and then wipe the machine with it.
- Do not use a hair dryer to dry the surface.
- Be careful when wiping the machine with denim and colored fabrics as the color of the fabric will stain the surface of the synthetic leather.
- Do not keep these parts in contact with plastic for a long time, as this will cause discoloration.
- Synthetic leather may be discolored, so when using hair dye, please cover the touched part with a towel.

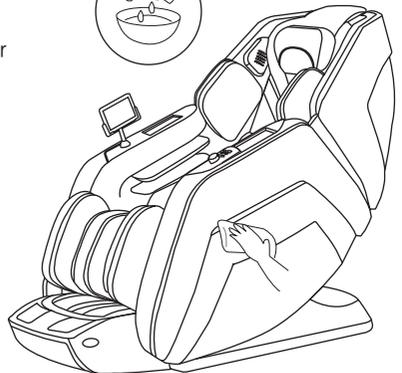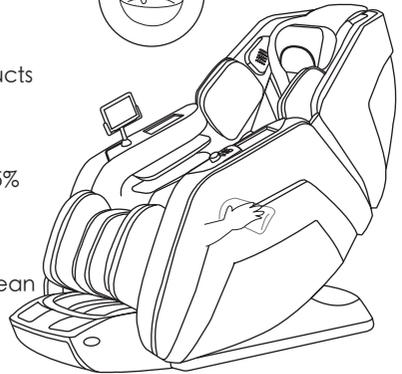

Clean the plastic part


Dip a soft cloth in water or hot water containing neutral washing, press dry, and wipe the plastic part of the machine.

- Do not use chemicals such as thinner, gasoline, alcohol, etc. to clean the unit.
- Wipe with a cloth soaked in clean water and thoroughly wrung out
- Be sure to wring out the cloth before wiping the remote control.
- Let it air dry naturally.

Troubleshooting

The following action sound occurs during use: Feeling, but this is due to the structure of the massage chair and is not abnormal.

- The sound of the massage wheel changed from "extend" to "retract";
- The sound and feeling of the massage wheel rubbing the wrinkle of the fabric worker during action;
- Action sounds of kneading, beating, etc.;
- The friction sound when the massage wheel moves;
- Action sound when moving up and down;
- Rotation sound of conveyor belt;
- Motor sound;
- The sound when the massage wheel changes from "squeeze" to "stretch";
- The sound when seated;
- The friction sound between the armrest and the cushion when the backrest is tilted;
- Rubbing sound between massage wheel and cloth;
- The working sound of the air pump when the airbag massage is operating;
- Exhaust sound when airbag massage operates;
- The workpiece sound emitted by the solenoid valve under the seat when the airbag massage is operating;
- Shaking of the legrest;
- The sound of shoulder extension and movement;
- Airbag action sound;
- The sliding sound of the foot massage part;
- When the backrest is tilted, the sound when the backrest rubs against the leather on the back cover becomes louder, and there may be a malfunction.
- In this case, please entrust an authorized service center for inspection and repair.

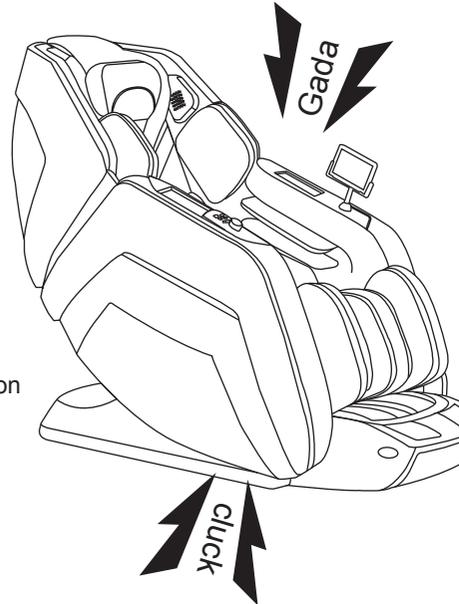

Troubleshooting

After finding the fault, first check and try the following methods.
If the fault persists after trying, please contact the dealer.

Question	Causes and how to deal with them
<p>During the process of making this machine, the massage wheel stops (the massage chair makes a "beep beep" sound, and all the display icons and buttons on the remote control flash and then go out.</p>	<p>During use, when the massage wheel encounters excessive external force, the massage wheel may stop moving for safety reasons. If all the marks and buttons on the controller start to flash, which is extremely abnormal, turn off the power switch, wait about 10 seconds, and then turn the power on again. If the massage wheel stops again, please adjust your sitting position so that your back is not completely on your back. (Pay attention to the precautions related to being guarded, especially when used by people weighing 200kg or more).</p>
<p>Can't tilt The calf cannot be raised or lowered (The sound of "beep beep" will be heard, and the running indicator of the remote control will turn off and stop running)</p>	<p>If the machine encounters an obstacle or is used with excessive force, the machine will stop working for safety reasons. When all the displayed contents and buttons are flashing, and the backrest or calf has stopped working, turn off the power of the machine once, and then "turn on" the power again after about 10 seconds, and then operate the machine.</p>
<p>Massage does not reach the shoulders or neck.</p>	<p>Because when you lie down on the massage chair, the detection is not in place, and it cannot be massaged. This situation is purely natural, not a malfunction of the body, and you can re-detect the shoulder width position.</p>
<p>This unit does not work at all ●Upper body (neck to waist) massage ●Calf and foot massage</p>	<ul style="list-style-type: none">● The power cord is disconnected;● The power switch of the machine is not turned on;● After pressing the lift button on the controller, no program is selected.
<p>Even if the warming function is turned off, the temperature will not go down</p>	<ul style="list-style-type: none">● Due to the structure of the heater, it will still feel warm due to residual heat for a while after heating.● Even if the warming function is turned off, continuing to massage the same part will increase the body temperature.
<p>The unit is damaged</p>	
<p>The power cord or power broadcast head is abnormally hot</p>	<p>To prevent accidents, be sure to contact an authorized service center.</p>

Product Specifications

Technical parameter

product name:	OPLUENZA Massage chair
Product number:	L36
Rated voltage:	220-240V~50/60Hz
rated power:	150W
Product Size:	161*76*116cm
Executive standard:	GB4706.1-2005 GB4706.10-2008
safety class:	Class I

The company reserves the right to improve the product and the manual. If there is any change, there will be no notice. The appearance of the product is subject to the actual product. This book has been carefully checked, but it does not rule out that there may be printing errors or misunderstandings in the content. Please understand, our company reserves the right of interpretation.